

Drugs of Abuse and Comprehensive Drug Analysis in Urine: Send Orange Screw-Cap Cup Emergency Centers and Inpatient Units

In order to have sufficient urine sample to perform drug confirmation testing, Beaumont Laboratory requests a **50-mL random urine specimen** (Minimum: 30 mL for adults, 20 mL for pediatrics) in an *orange screw-capped container* (*PeopleSoft 2100304, preferred*) or other sterile collection container. The orange screw-cap sterile specimen cups should be labeled with an Epic Beaker bar-coded sticker, with collection date and time, and closed tightly.

These containers may be reliably transported in pneumatic tube systems:



This container is to be sent to the lab at each Beaumont location for all drugs of abuse testing in urine, including:

Drugs of Abuse with Confirmation, Urine (LAB6862)

Drugs of Abuse without Confirmation, Urine (LAB8506) – confirmation performed upon request
Comprehensive Drug Analysis (LAB7162)

Drug screening is performed at all sites and confirmation testing at the Royal Oak lab.

If you have questions, please contact your Beaumont Laboratory Customer Service Department:

- Farmington Hills, Grosse Pointe, Royal Oak, and Troy at **800-551-0488 or 248-551-1155, Option 5**
- Dearborn, Taylor, Trenton, and Wayne at **800-245-3725, Option 1**

Laboratory Test Directory: <http://beaumontlaboratory.com/test-lab-directory>

Date submitted: May 5, 2021
Submitted by: Steven M. Truscott, PhD, DABCC, Technical Director, Special Chemistry, STAT Lab, Point of Care Testing
Elizabeth Sykes, MD, System Medical Director, Chemistry