

ALGORITHMS FOR REFLEX TESTS

CLINICAL PATHOLOGY

ABO Grouping:

- If patient appears to be group A or AB with unexpectedly positive reverse typing, then ...
 - Reflex testing: special antigen typing for A1 antigen, and antibody identification
 - Process followed for group AB patients also.
- If patient appears to be group B with unexpectedly positive reverse typing, then...
 - Reflex testing: antibody identification.

Acid Elution: (Royal Oak)

- The Hematology Laboratory performs the test when the Flow Cytometry Laboratory is closed. Specimen is then forwarded to Flow Cytometry for the Fetal RBC Assay.

AFB Culture:

- Includes molecular testing M.tb complex and rifampin resistance as indicated on sputum and tracheal aspirates ONLY.

Amniotic fluid AFP:

- If elevated AFP MoM or there is a specific clinical history, sample sent out to Mayo Medical Labs for repeat AFP and if necessary, acetylcholinesterase testing.

Amniotic fluid algorithm:

- If the Lamellar Body Count (LBC) is consistent with fetal lung immaturity, sample is sent to Mayo Medical Labs for an L/S-PG. If the LBC indicates maturity, no more testing performed.

Anti-Nuclear Antibody (ANA) evaluation:

- Run ANA screen by immunofluorescence
- If greater than cut-off, immunofluorescence microscopy performed - pattern and titer reported.

Anti-Nuclear Antibody (ANA) evaluation with reflex:

- Run ANA screen by immunofluorescence
- If result is positive, report titer and pattern AND reflex to specific autoantibodies for SSA, SSB, Smith, RNP and dsDNA.

Antibody Identification:

- If a new antibody specificity is identified, then...
 - Reflex testing: Special Antigen Typing of patient's RBCs for each specificity (may result in multiple charges).
- If autologous control is positive during antibody identification, then...
 - Reflex testing: Direct Antiglobulin Test x1 (DAT X3 polyspecific, monospecific IgG and C3d done if poly is positive).
- If prenatal patient has or has had clinically significant antibody identified, then...
 - Reflex testing: Antibody Titer q 3-4 weeks. If multiple clinically significant antibodies are present, then a titer is performed for each specificity resulting in multiple charges.
- If antibody activity includes a broadly reactive cold or warm antibody interfering with determination and identification of alloantibodies or crossmatches, then...
 - Reflex testing may include: Adsorptions, Pretreat RBC w/ Enzymes and/or Pretreat Serum by Dilution and/or Pretreat Serum by Diff, Red Cell Adsorptions and Antibody Identification, antigen

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

typing of the patient. Multiple charges may be generated for each sample depending upon complexity of the serological problem.

- If an Antibody Identification is ordered and the patient has no previous history of antibody, then...
 - Indirect Coombs (Antibody Screen) is performed first.
- If an Antibody Titer is ordered, and the patient has not had antibody identification performed within the last month, then...
 - Antibody Identification is performed first.

Antibody Screening (Indirect Coombs):

- If patient has no previous history of a positive antibody screen, then...
 - Reflex testing: Antibody Identification. Multiple antibody identification panels may be performed in order to elicit antibody specificity.
- If patient has previous history of a positive antibody screen, and ...
 1. Time elapsed since the last antibody identification is
 - greater than 1 month for an obstetrical patient, or
 - greater than 3 months for a non-obstetrical patient, OR
 2. Reactivity occurs with RBCs known to lack the corresponding antigen, OR
 3. Reactivity has increased in strength since the last antibody screen, OR
 4. Patient has a previous history of a WAA (warm auto-antibody) that is currently reactive in the antibody screen OR
 5. Patient is incompatible when crossmatched with antigen negative RBCs, then...
 - Reflex testing: Antibody Identification. Multiple antibody panels may be performed in order to elicit antibody specificity(ies)
- If patient has or previously has had a positive antibody screen (not due to Rh Immune Globulin administration) then...
 - Reflex testing: two (2) RBCs are crossmatched in addition to crossmatch orders.
- If obstetrical patient has history of or current clinically significant antibody, then...
 - Reflex testing: cord blood survey consisting of Direct Antiglobulin Test, Typing ABO, Typing RHO (D), Special Antigen Typing. Multiple charges for Special Antigen Typing if mother has multiple clinically significant antibodies.

Bacterial Antigen Detection (CSF specimens):

- The bacterial antigen test on a CSF specimen will not be performed if the WBC count of the CSF is less than 10 mil/L.
- Reflex to include a culture (if not ordered) - College of American Pathologists guideline

Barbiturate screen including serum quantitation:

- Urine screened for barbiturates.
- If positive, urine confirmation and serum quantitation performed

Bone Marrow:

- Ancillary testing, including special stains, immunoperoxidase stains, flow cytometry, cytogenetics, FISH and molecular testing will be performed for diagnostic purposes, on bone marrows and other hematopoietic specimens, as determined necessary by the pathologist to render a diagnosis.
- Cytogenetics, FISH and molecular testing will be performed for prognostic and treatment purposes, based on DMT guidelines (Appendix I).

CK-MB:

- When a CK-MB is ordered, a total CK is performed. If the total CK in males is less than 100 U/L or in women is less than 80 U/L, the CK-MB is NOT performed. If the CK-MB is not performed, the total CK is still reported.

Clostridium difficile CDH/Toxin NAA testing:

- In cases where GDH is negative and Toxin A/B is positive, NAA testing is performed.

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

Complete Blood Count (CBC):

- If results are abnormal and meet criteria for requiring pathologist review, add differential. The slides, instrument printout and lab printout are saved for pathologist review.

Complete Blood Count with Differential (CBC/diff):

- If results are abnormal and meet criteria for requiring pathologist review, the slides, instrument print out, and lab print out are saved for pathologist review.
- If sickle cells are found on a differential, a sickle cell screen is performed if no previous history.
- If suspect microorganisms are seen on smear, refer to Microbiology for confirmation.

Comprehensive Drug analysis:

- All drugs found on immunoassay screen (amphetamine, cocaine, cannabinoids, barbiturates, benzodiazepines, opiates, methadone, PCP) are confirmed by an alternate method

Crossmatching:

- If patient has clinically significant antibody(ies), then...
 - Reflex testing: Unit Antigen Screen with antisera. Multiple charges generated (# of units tested x number of antigens tested for)...
- If patient with a previous history of clinically significant antibody has incompatible crossmatches with units known to lack corresponding antigen(s), then...
 - Reflex testing: Antibody Identification
- If patient has incompatible crossmatches due to broadly reactive warm or cold auto or allo antibody, then...
 - Reflex testing: Unit Antigen Screen with patient sera, unit antigen screen with reagent anti-sera, phenotypically similar matched units or least incompatible. Multiple charges will be generated equal to number of units tested to locate compatible or least incompatible.
- If patient has a history of Sickle Cell Anemia/Thalassemia then...
 - Units will be confirmed Hemoglobin S negative (additional charge per unit), and antigen negative (C, E, Kell) per patient antigen typing. Multiple charges generated (# of units tested x number of antigens tested for)

Cryoglobulins:

- If a positive cryoglobulin is detected, cryoglobulin characterization and quantitation will be performed automatically for Farmington Hills and Troy samples and on request for Royal Oak and Grosse Pointe samples.

Cryptococcal Antigen Test (CSF only):

- Reflex to include a culture (if not ordered) - College of American Pathologists guideline

CSF cell count and differential:

- All CSF specimens with WBC counts within the normal range (≤ 5 WBC/mcL) will have a CSF scan differential performed. RBCs, PMNs and mononuclear cells will be reported if present.
- If the CSF WBC count is above the normal range (> 5 WBC/mcL), a differential will be performed.
- If results are abnormal and meet criteria for requiring pathologist review, the slides and lab print out are saved for pathologist review.
- If suspect microorganisms are seen on the smear, refer to Microbiology for confirmation.

Direct Antiglobulin Test:

- If Direct Antiglobulin Test is positive for IgG, and patient is baby of mother with clinically significant antibody, then...
 - Reflex testing: Antibody Identification on eluate from baby's RBCs.
- If Direct Antiglobulin Test is positive on post transfusion reaction specimen, then...
 - Reflex testing: Antibody Identification on eluate from patient's RBCs.
- If a Direct Antiglobulin Test is ordered on a patient > 4 months of age, then...
 - DAT is performed using polyspecific AHG. If positive then reflex test: 2 additional DATs IgG and C3 (monospecific).
 - If Direct Antiglobulin Test is positive within two weeks of transfusion, then...

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

- Reflex Testing: Transfusion Reaction Evaluation if deemed necessary by pathologist

Drugs of Abuse screen (amphetamine, cocaine, cannabinoids, barbiturates, benzodiazepines, opiates, methadone, PCP):

- All positive results are confirmed by an alternate method.

DRVVT (screen and confirm):

- A dRVVT screen is performed. If screen is abnormal, a confirmation test is performed on the patient and a control. Normalized ratio of screening test to confirmation test is calculated and reported.

Fetal Cell Screening:

- If post-partum mother is Rh(D) negative and baby is Rh(D) positive, then...
 - Reflex testing: Fetal Cell Screen
- If the fetal cell screen is positive, mother or baby is weak D positive, OR baby's Rh type cannot be determined (e.g. no specimen, invalid Rh testing results), then...
 - Reflex testing: Fetal RBC Assay (FMH) ordered by Blood Bank.

Fluid cell count and differential:

- Body fluid differentials will be performed when the WBC count is >5 mCL.
- If results are abnormal and meet criteria for requiring pathologist review, the slides and lab print out are saved for pathologist review.
- If suspect microorganisms are seen on the smear, refer to Microbiology for confirmation.

Group A Strep Screen:

- If positive, no additional testing is performed unless specifically requested
- If negative, reflex to Group A Strep by Nucleic Acid Amplification for ages < 16 years.

Group B Strep Screen (*Streptococcus agalactiae*):

- A vaginal/rectal screen reflex to include susceptibility testing.

HBsAg:

- Confirmed when result meets criteria of >1.00 - <50.00. If >50, then not required.

HCG Serum Qualitative:

- For Troy only if hCG Serum Qualitative test is positive, a quantitative hCG is performed and reported.

HCV antibody testing for inpatient "source":

- All positives are tested for HCV RNA Quantitation by PCR

Hemoglobinopathy evaluation:

- Run chromatography (Bio Rad Variant)
- If abnormal hemoglobin detected, run electrophoresis

Hexagonal Phase Phospholipid (screen and confirm):

- A hexagonal screen is performed. If the screen is abnormal, a confirmation test is performed on the patient. The difference between the screen and confirmation is calculated and reported.

HIV antigen/antibody testing:

- HIV 1/2 antibody and p24 HIV-1 antigen test used as initial screen. If positive, additional testing performed using BioRad Multispot.
- If Geenius is negative or equivocal, sample(s) sent to outside lab for HIV-1 RNA and if appropriate HIV-2 DNA/RNA testing.

HLA: HLA antibody identification ordered and performed when the HLA (cytotoxic) antibody screen is positive.

HLA B27:

- If flow result is in equivocal range, report as "equivocal" and refer for Class 1 typing for disease

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

association.

IgG subclasses: Total IgG included

Inhibitor coagulation screen:

- PT and PTT performed. At Royal Oak, if the PTT is prolonged, a thrombin time is performed to rule out drug effect. Farmington Hills, Royal Oak and Troy perform heparin absorption as needed. If PT is prolonged or the PTT is prolonged without evidence of heparin, then 1:1 mixing studies are performed both immediately and after 120 min incubation at 37°C. Additional testing and/or pathologist interpretation is performed as needed.

Lupus Anticoagulant:

- If this is ordered it reflexes to "Coagulation consult for anti-phospholipid antibody."

Lyme Antibody:

- All positive or equivocal results will be sent out for Western blot testing.

Molecular Genotyping:

- Molecular Genotyping (send out test) will be performed (additional charge generated):
 - When a patient has 3 or more Alloantibodies OR
 - Patient has Sick Cell/Thalassemia OR
 - Patient is a Warm Autoantibody patient OR
 - Patient being treated for Multiple Myeloma (Daratumumab)

Peripheral Smears with Microorganisms:

- Reflexed to Microbiology for further evaluation.

Protein C:

- Protein C activity is performed. If abnormal and the patient is not on Coumadin, it is verified with a protein C antigen.

Protein S:

- Protein S activity is performed. If abnormal and the patient is not on Coumadin, it is verified with a protein S free and total antigen.

Serum monoclonal gammopathy evaluation:

- Run SPE and complete immunofixation
- If new abnormality or monoclonal obscured by other proteins, one or more immunoglobulin quantitations performed.
- Immunoglobulin quantitations also performed at discretion of pathologist (e.g. hypogammaglobulinemia).

Serum protein electrophoresis: [Royal Oak: for in-patients only. Grosse Pointe, Farmington Hills and Troy: for in-patients and outpatients]

- If a significant abnormality is detected on serum protein electrophoresis and the pathologist or clinical chemist determines that an immunofixation is appropriate, an immunofixation is performed.

Sickle Cell:

- If sickle cell screen is positive, Hemoglobinopathy evaluation is reflexed.

Smear Blood Parasite → Includes Malaria antigen screen when indicated

Stool Culture → Includes testing for enterohemorrhagic *E. coli*

Trichomonas Culture → Includes direct examination when appropriate

TSH with reflex to free T4:

- TSH performed

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

- If TSH falls outside specified limits, a free T4 is automatically performed.

Urinalysis:

- Dipstick testing is performed; if positive result encountered (other than urobilinogen), automated and/or manual microscopy is performed.

Urine protein electrophoresis:

- Run UPE and kappa/lambda immunofixation
- Run complete immunofixation at Pathologist's discretion or if new abnormality

RPR:

- All positives tested by TP-PA (Treponema pallidum particle agglutination) – Effective 01/03/2017

Virus Culture: → Specific testing determined by site, source or suspect agent.

Wet Prep → Includes *Trichomonas* Antigen testing.

Yeast – Antimicrobial sensitivity testing (AST) will be performed on all yeast recovered from blood cultures.

ANATOMIC PATHOLOGY

In the Anatomic Pathology laboratory, pathologists use their judgement to order and examine additional studies on submitted specimens if they deem them medically necessary to render a diagnosis. In these situations, pathologists are acting in the capacity of a consultant in the care of the patient. This includes but is not limited to ordering special stains, decalcification of tissue, immunoperoxidase stains, microbiology cultures on tissue specimens, electron microscopy, and flow cytometry on certain tumors if indicated.

The Anatomic Pathology reflex test list includes tests that Beaumont Laboratory performs if a specimen meets the reflex criteria listed and tests that are often useful for the diagnosis and prognosis of the patient. Treating providers may decline the reflex testing by indicating that on the laboratory requisition or by contacting Beaumont Laboratory's Customer Service Department.

BCR/ABL1 for t(9;22)(q34;q11.2) detection

- When a BCR/ABL1 Dual Color Dual Fusion Translocation probe produces an atypical signal pattern (1R1G1F in >23% of nuclei) indicating the arginosuccinate synthetase (ASS) gene on chromosome 9q34 may be deleted, reflex testing to the BCR/ABL1 + 9q34 Tri-Color Dual Fusion probe should be performed to determine if there is an ASS gene deletion present.

Estrogen and Progesterone Receptor Status (IHC)

- Performed on all newly diagnosed invasive and recurrent/metastatic breast cancers to assess responsiveness to endocrine therapy

Fluorescence *in situ* hybridization (FISH) Panels

- All FISH panels are performed according to the DMT guidelines (Appendix I).

Her2-Status (IHC and FISH)

- Performed on all newly diagnosed and recurrent invasive breast cancers, and metastatic tumors, to guide the decision to pursue Her2-targeted therapy.
 - If the Her2 test result on a core biopsy specimen of a primary tumor is **negative** and the tumor is grade 3; or a small amount of tumor in the core biopsy; or resection specimen contains high-grade carcinoma morphologically distinct from that of the core specimen; then repeat testing on an excisional specimen
 - If the Her2 test result on a core biopsy specimen of a primary tumor is **positive** and the tumor is histologic grade 1 that is of the following histologic types: infiltrating ductal or lobular and is ER- and PgR-positive, or tubular, or mucinous, or cribriform, or adenoid cystic carcinoma, then repeat testing on an excisional specimen

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

- Performed on all newly diagnosed gastric, gastroesophageal junction adenocarcinomas and their recurrences/metastases
- Performed on all newly diagnosed distant metastases of colorectal carcinoma.

IDH1 (Isocitrate Dehydrogenase) (IHC)

- Performed on all infiltrating gliomas for diagnostic and prognostic purpose

MGMT (O6-Methylguanine-DNA Methyltransferase (**MGMT**)) Methylation Status

- Biomarker in pediatric and adult patients with glioblastoma, for diagnostic and therapy guiding purpose

Mismatch Repair Analysis (IHC)

- Performed on all newly resected colorectal carcinomas and endometrial carcinoma as a screen for Lynch syndrome
- Performed on any metastatic carcinoma (from any primary organ site) for possible immunotherapy.

Aggressive B-Cell Lymphoma Panel

- Performed according to aggressive B-cell lymphoma FISH diagnostic algorithm (DMT guidelines).

PDL-1 (Clone 22C3)

- Performed on all newly diagnosed metastatic non-small cell carcinoma (NSCLC subtypes: adenocarcinoma, squamous cell carcinoma, adenosquamous carcinoma, large cell neuroendocrine carcinoma, NSCLC favor squamous cell/adenocarcinoma, or NSCLC-NOS).
- Performed on all metastatic carcinoma in which the possibility of a NSCLC cannot be entirely excluded.

p16

- Immunohistochemistry (IHC) testing performed on all newly diagnosed head and neck squamous carcinomas arising in the oropharynx (Base of tongue and tonsils)
 - IHC testing performed on squamous carcinoma presenting as a lymph node metastasis of unknown primary site
-

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

Appendix I – DMT Guidelines

BEAUMONT HEALTH GUIDELINES FOR CYTOGENETIC/MOLECULAR TESTING OF HEMATOPOIETIC MALIGNANCY

Revised Date: Oct 1, 2020

Effective Date: Oct 14, 2018

With the expanding discovery of new genetic markers, it became imperative to have a test protocol in place to ensure compliance with NCCN guidelines and WHO guidelines. Thus, a multidisciplinary committee was formed by Beaumont Laboratory to standardize cytogenetic and molecular testing for hematopoietic malignancy across the hospital system. This Laboratory Bulletin summarizes the current testing protocol and ancillary tests that will automatically be ordered by the pathologists in all new diagnoses of hematopoietic malignancy.

B Lymphoblastic Leukemia/Lymphoma	<ul style="list-style-type: none"> Karyotype ALL FISH Panel [4,10 trisomy, t(9;22), 11q23 rearrangement, t(12;21), t(1;19), IGH/IL3 for t(5;14), iAMP21, ABL1, ABL2, PDGFRB per Children's Oncology Group protocol] RT-PCR for BCR-ABL1 Send out for Phil-like ALL FISH Panel testing (MAYO), at ordering physician request only
Acute Myeloid Leukemia, including evolving AML	<ul style="list-style-type: none"> Karyotype Limited AML FISH Panel [CBFB gene rearrangement for inv16/t(16;16), p53, t(15;17), MLL, t(9;22), NUP98] on all Expanded FISH panel upon request to also include t(8;21) and 3q26 rearrangement Next Generation Sequencing - Myeloid Panel FLT3
Myelodysplastic Syndrome	<ul style="list-style-type: none"> Karyotype Next Generation Sequencing - Myeloid Panel FISH only upon request or at discretion of pathologist or if traditional karyotype obtained <20 metaphases [MDS FISH Panel: trisomy 8,del(20q),monosomy 5/del(5q),monosomy 7/del(7q),3q26 rearrangement, monosomy 13/del(13q), MLL,p53]
Myeloproliferative Neoplasms	<ul style="list-style-type: none"> Karyotype FISH or RT-PCR for BCR-ABL1 JAK2 V617F, if negative CALR, MPL (ET and MF) and JAK2 Exon 12 (PV)
Myelodysplastic/Myeloproliferative Neoplasms including Chronic Myelomonocytic Leukemia	<ul style="list-style-type: none"> Karyotype FISH or RT-PCR for BCR-ABL1 JAK2 V617F, if negative CALR and MPL Eosinophilia FISH Panel [FIP1L1/PDGFR, PDGFRB, FGFR1 and JAK2 rearrangements] MDS FISH only upon request or at discretion of pathologist or if traditional karyotype obtained <20 metaphases [MDS FISH Panel: trisomy

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.

Laboratory Bulletin

	8,del(20q),monosomy 5/del(5q),monosomy 7/del(7q),3q26 rearrangement, monosomy 13/del(13q), MLL,p53]
Eosinophilia associated with myeloid/lymphoid neoplasms	<ul style="list-style-type: none"> Eosinophilia FISH Panel [FIP1L1/PDGFR, PDGFRB, FGFR1 and JAK2 rearrangements]
Plasma Cell Myeloma	<ul style="list-style-type: none"> Karyotype Myeloma FISH Panel [CDKN2C/CKS1B for chromosome 1q rearrangement, CEP3/7 for hyperdiploidy, monosomy 13/del(13q), p53, IGH/CCND1 with reflex to IGH/FGFR3, IGH/MAF, and IGH/MAFB]
Chronic Lymphocytic Leukemia/ Small Lymphocytic Lymphoma	<ul style="list-style-type: none"> CLL FISH Panel [MYB/CEP6, trisomy 12, p53,ATM,monosomy 13/del(13q), IGH/CCND1] Karyotype
Diffuse Large B-cell Lymphoma	<ul style="list-style-type: none"> FISH for double hit lymphoma all cases [MYC and IGH/MYC; Reflex to BCL2 and BCL6 if MYC rearranged] Karyotype optional
Burkitt Lymphoma	<ul style="list-style-type: none"> [FISH for IGH/MYC with reflex to IGK/MYC and IGL/MYC]
Hairy Cell Leukemia	<ul style="list-style-type: none"> BRAF V600E (send out)
Lymphoplasmacytic Lymphoma	<ul style="list-style-type: none"> MYD88 L265P CXCR4 at request of ordering physician for patient's being considered for ibrutinib
Gastric Malt Lymphoma	<ul style="list-style-type: none"> H. pylori testing (also should be performed in gastric DLBCL) If positive for H. pylori; FISH for t(11;18) will be performed
T-cell leukemia/lymphoma including T-cell Prolymphocytic Leukemia	<ul style="list-style-type: none"> Karyotype T-cell leukemia/lymphoma FISH Panel [TCL1 (14q32), TCRAD (14q11), TCRB (7q434), and TLX3 (5q32) rearrangements; chromosome 8 abnormalities; del(9p)]
Peripheral T-cell Lymphoma and Primary Cutaneous CD30 Positive T-cell Lymphoproliferative Disorders	<ul style="list-style-type: none"> FISH for DUSP22 if ALK negative ALCL consideration (paraffin only)
Other lymphomas	<ul style="list-style-type: none"> Additional cytogenetic and molecular testing is useful under certain circumstances and will be performed at discretion of pathologist; nothing routinely performed or required NHL FISH Panel (IGH/CCND1, IGH/BCL2, MALT1, MYC, BCL6)

Date submitted: October 1, 2020

Submitted by: Chiefs of Pathology

Mitul Amin, MD, Royal Oak, Anatomic Pathology
 Muhammad Arshad, MD, Taylor, Trenton, Wayne
 Peter Millward, MD, Royal Oak, Clinical Pathology
 Vaishali Pansare, MD, Grosse Pointe, Troy
 J Troy Powers, MD, Dearborn
 John Pui, MD, Farmington Hills

Printed copies of this document need to be confirmed with online version to verify that it is up-to-date before use.