

GREINER TUBE VISUAL CROSSWALK CHART

Previous Vendor - BD		New Vendor - Greiner	Generic Name and notes	Tube Code - South Sites only
366703 366408 	→	454241  	Discard Tube	
367886 367884 	→	456088 454029 	Lithium Heparin When spun, creates Plasma	GL
367871 367878 	→	454030 456028 	Sodium Heparin When spun, creates Plasma Used mainly in Flow and Cytogenetics	GN
367812 366430 367815 367814 	→	454204 455092 456089 	No additive clot tube When spun, creates Serum Used mainly for routine Chemistry and Toxicology	R7 or RN
367863 367861 367856 	→	454209 	EDTA When spun, creates Plasma Used mainly for Hematology. Some Molecular, Special Chem, and Flow	LF, LV, or L7
367841 	→	454428 	EDTA 2 mL When spun, creates Plasma Used mainly for Hematology. Some Molecular and Special Chem	LF, LV, or L7
367921 	→	454238 	Sodium Fluoride/Potassium Oxalate 2 mL When spun, creates Plasma Used mainly for Glucose testing	GY
367922 367925 	→	454297 456020 	Sodium Fluoride/Potassium Oxalate When spun, creates Plasma Used mainly for Glucose testing	GY
363080 	→	454322 	Sodium Citrate 1.8 mL When spun, creates Plasma Used mainly for Coag	B
363083 	→	454334 	Sodium Citrate 2.7 mL	B
367899 	→	456279 	Pink Top EDTA Used for Blood Bank	P
367983 367986 	→	454228P 456018P 	Gold Top/SST When spun, creates Serum Used mainly for routine Chemistry and Special Chemistry. Some Toxicology	S7
367960 367962 	→	454247P 456087P 	Lithium Separator/Troponin Tube/PST When spun, creates Plasma Used mainly in Stat lab for Troponin and stat Chemistry	GT
368380 		456016 	Royal Blue No Additive When spun, creates Serum Used for Trace metal testing	DB
364951 	→	454486 	Urine Culture tube	UC
364966 	→	450251 	Urine Transfer Straw	
364953 	→	453031 	Urine Culture tube with transfer straw	UC