

# URINE SPECIMEN PROCESSING

## STEP 1: Collect Urine




URINE CUP USED FOR TRANSFER to Tubes ONLY



Dispose of any unused urine in toilet and discard cup in trash

## STEP 2: Fill Tubes

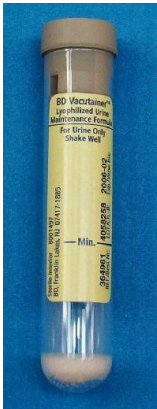

C&S



UA



Discard Blue Cap into Sharps Biohazard bin



NOTE: If a Urine Culture or UA Specimen is sent along as an "Extra "specimen (no orders), the specimen tube MUST be labeled with:

- patient name
- MRN
- Date/time of collection
- Collector ID

## Step 3: SEND TO LAB